

ORALMEDIC®

Eliminates mouth ulcer pain in seconds!

One application of **ORALMEDIC®** stops the pain and discomfort of a mouth ulcer in seconds. A single use prefilled **ORALMEDIC®** swab delivers the amount of liquid material required to treat one typical mouth ulcer. It works by forming a protective barrier over the ulcer to provide optimum healing conditions for natural healing.

ORALMEDIC® is for the treatment of ulcerations in the tissue of the mouth. It is not intended or effective in the treatment of cold sores, fluid-filled blisters or vesicles in the mouth or on surface of the lips.

WARNING: Do not use if allergic to sulphur.

Instructions:

Important - Please read carefully



1. Dry affected area using the cotton drying bud.



2. Hold the applicator at arms length, away from the eyes, with coloured ring up. Gently bend the coloured ring to one side until it snaps. **ORALMEDIC®** will flow to the opposite end.



3. Apply to the affected area for 5-10 seconds. Use a rolling motion to completely cover the entire ulcer bed and ulcer rim. A 'stinging' sensation is experienced immediately upon application.



4. **Rinse mouth thoroughly with water and spit out.**

5. Dispose of swab in the bin.

An effective treatment is evident when a "white patch" is seen in and around the ulcer. This white patch may flake off during the healing period.

Note: If the ulcer pain remains after rinsing with water, it is an indication that some part of the ulcer was not covered. Repeat application immediately, one more time only, following the directions above. Thereafter do not retreat again.

Important Safety information: Do not use if allergic to materials that contain sulphur in any form. Do not use if you are pregnant, or on children under the age of 12. Avoid contact with the eyes. If eye exposure occurs, immediately remove any contact lenses, irrigate eyes for at least 15 minutes with water and seek immediate medical advice.

May cause local irritation or mild swelling when applied. If excess irritation occurs during use, rinse thoroughly with plenty of water. If condition persists, consult your doctor or dentist.

Avoid contact with clothing as it may discolour or damage.

Keep out of reach and sight of children.

Store at room temperature in a dry place.



If used as directed the product cannot be reused. After the swab is activated and liquid is applied, the oral cavity is rinsed with water and spat out, with the used swab properly disposed of.

Ingredients: HYBENX® (Hydroxybenzenesulfonic Acid, Hydroxymethoxybenzenesulfonic Acid, Sulphuric Acid, Water, FD&C Red 40).

This pack contains: 2 single use applicators each containing 0.2ml of HybenX solution. 2 drying buds.



© EPIEN Medical, Inc. 4225 White Bear Parkway, Ste 600, St.Paul, MN 55110 USA www.epien.com

HybenX® is a registered trademark of EPIEN Medical, Inc.



QNET BV, Hommerterweg 286, 6436 AM Amstenrade, The Netherlands.

Text prepared 02/07 rev 06/11 Form ENG_L01.

whiteTM

communications group



C



M



Y



K

Brand	Oralmedic
Component	Leaflet
Country	UK
WTD Job No.	14203
SKU No.	XXXXX
Dimensions	mm
Text size (pts)	7 pts
Printer	TBA
Date	16/02/12

White Communications Group March House
44 Leigh Road Eastleigh Hampshire SO50 9DT
t 02380 620088 www.whitecommsgroup.co.uk

Version: 6